



Renew[®] Inserts



Confidence & Control. **In Comfort.**

**Innovative protection from
Accidental Bowel Leakage**






Renew[®] Inserts

Confidence & Control.
In Comfort.

Renew Inserts softly and comfortably fit the body, sealing the rectum from the inside to help prevent Accidental Bowel Leakage (ABL). They provide reliable and discreet internal protection that allows your patient the opportunity to live a normal, active life.

A single Renew Insert can be worn day and night, until the next bowel movement, or changed as desired. Simple and easy to use, each insert is individually packaged with a handy, disposable finger-tip inserter.

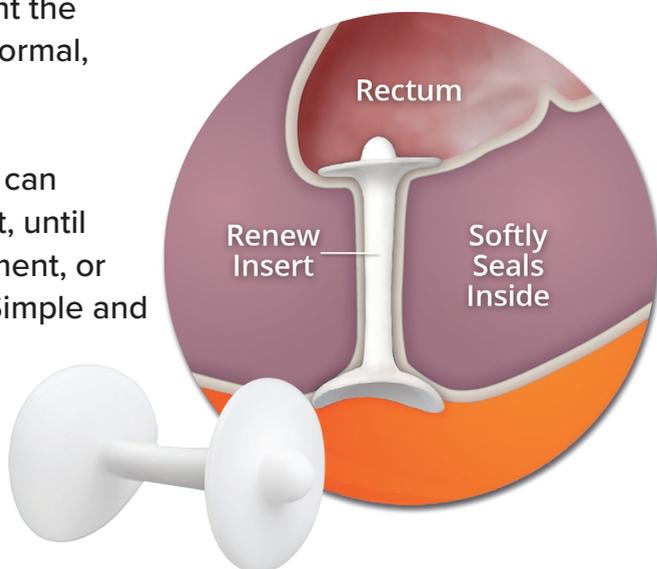

Frequently Asked Questions **by patients:**



Are Renew Inserts comfortable to insert and wear?

Clinical studies show Renew Inserts are soft, fit the contours of the body and are comfortable to wear.

Will I feel the Renew Insert in place?

Most people can feel the Renew Insert in place when they first try them. This feeling will usually pass with use.

How do I know when the Renew Insert is properly in place?

When the Insert is properly in place, the top disc will rest just inside the rectum and the bottom disc, just outside the anus. Your patient should not feel any discomfort.

How long can I wear a Renew Insert?

Renew Inserts may be worn continuously, day and night. A single Insert can be worn until your next bowel movement, or changed as desired. After one is expelled, your patient may replace it with a new Insert.

How do I know what size Renew Insert to use?

Initially, we recommend trying the Regular size Insert. If your patient finds the Insert comes out at unexpected times or if they experience leakage, try using a Large size Insert.

What are the recommended insertion positions?

Your patient may want to sit on the toilet or try standing with one foot up on the toilet, or squatting. They should use whichever insertion position feels most comfortable for them.

Can a Renew Insert get lost inside my body?

No. Clinical research has shown that Renew

Inserts come out with during their next bowel movement. In the unlikely event a Renew Insert moves into the anus, it will be naturally expelled at that time.

What if the Renew Insert comes out when I go to the toilet and pass wind, or urinate?

The Renew Insert is designed to be expelled with a bowel movement, but occasionally may come out if your patient is on the toilet, passing wind, or urinating. If this happens, they can simply replace it with a new Insert.

Will I be able to pass wind (gas) with the Renew Insert in place?

Yes. The Renew Insert still allows for the passage of wind (gas). If your patient is using a Large size Renew Insert and feel discomfort from wind (gas), try a Regular size Insert.

Can I bathe and be physically active when I'm wearing my Renew Insert?

Yes. Your patient can do all of their day to day activities while using a Renew Insert.

How often should I use the Insert?

Initially we suggest to try to use the Renew Insert daily, to become accustomed to wearing them. Usage can vary from person to person, based on their lifestyle, symptoms and condition.

Will I always have to use the Renew Insert?

This will depend on their personal symptoms and/or condition. You should be able to assist with ongoing assessment for your use of Renew Inserts.

Is the Insert available on the NHS?

Yes. Inserts are available on prescription, in boxes of 30 of your preferred size.

Confidence & Control. In Comfort.

Functional Causes

- Cancer
- Traumatic Childbirth
- Irritable Bowel (IBS/IBD)
- Post radiation Therapy
- Post Transanal irrigation
- Following surgery to the anal sphincter/pelvic floor

Neurogenic Causes

- Cauda Equina
- Multiple Sclerosis (MS)
- Parkinson's Disease
- Cerebrovascular Accident
- Diabetes



Description	Quantity	Product Code	PIP Code
Regular Insert	30	FG730	393-6895
Large Insert	30	FG731	393-6903
Starter Pack	2 (1 regular, 1 large)	FG716	

