

WARNINGS

Talk to your doctor if you have symptoms of erectile dysfunction (not being able to get or keep an erection that allows you to have sex) in order to prevent a delay in discovering and treating the cause.

Do not use this device if:

- You have an abnormally formed penis.
- You have been advised not to have sexual activity.
- You have sickle cell anemia or trait, leukemia, tumor of the bone marrow or other conditions that change blood clotting and may result in bleeding or prolonged erections.
- Do not leave the VenoSeal™ on your penis for more than 30 minutes. Do not fall asleep wearing the device. Leaving the VenoSeal™ on too long may cause permanent injury or damage to your penis.
- Allow at least 60 minutes between uses of VenoSeal™ device. Using adjustable Constriction Loops too often may cause bruising at the base of the penis (where the penis is joined to the body).
- Talk to your doctor if any complications occur in using this device. Stop using the device if complications continue.

PRECAUTIONS

- Ask your doctor about using the VenoSeal™ if you are taking anticoagulant (blood thinning) medicine; if you have a penile implant; or if you have Peyronie's disease, leukemia or other blood disorders where bleeding or clotting may be a problem. The effects of the VenoSeal on the components of implanted penile prostheses have not been adequately investigated. Therefore, if you have a penile prosthesis, consult your physician prior to use.
- Do not use the VenoSeal™ under the influence of drugs or alcohol. These substances may reduce your sensitivity to pain or impair your ability to follow directions properly.
- Do not use VenoSeal™ if you cannot sense pain in the penis. Pain may be a warning sign that the device is causing injury.
- Do not use VenoSeal™ if you do not have adequate use of your hands to easily remove device.
- Tension rings (including the VenoSeal™) are not a means of birth control. Do not rely on tension rings to avoid pregnancy.
- Always use the lowest amount of tension necessary to maintain the erection. Do not over-tighten the VenoSeal™ ring. Too much tension can cause bruising at the base of the penis.

POSSIBLE SIDE EFFECTS

Prolonged Erection/Priapism:

- Contact your doctor immediately if you experience an erection that lasts more than 4 hours. Prolonged erection or Priapism is a serious condition that requires medical attention to prevent permanent damage to the penis.

OTHER SIDE EFFECTS

- Pain or redness of the penis can occur as a result of using this device.
- Permanent injury to the penis can result if the device is either worn too tight or for more than 30 minutes.

CARE AND REPLACEMENT OF THE VENOSEAL™ ADJUSTABLE CONSTRICTION LOOP

- Care of the VenoSeal™: If you use lubricant for sexual activity, use only water based lubricants. Oil based lubricants may decrease the performance of VenoSeal™.
- VenoSeal™ can be used more than once. Clean after use with warm water and ordinary hand soap. Keep VenoSeal™ dry and store at room temperature after it is cleaned.
- Replacement: Repeated use of the VenoSeal™ device may result in wearing out the synthetic rubber top. Replace VenoSeal™ after 6 months of use or when it shows signs of wear such as cracking.



Timm Medical Technologies, Inc.
2111 W. Wyatt Earp
Dodge City, KS 67801
www.timmmedical.com

VenoSeal™

Adjustable Constriction Loop for Reliable Erections

- Helps maintain erections for up to 30 minutes
- Effective, affordable, and easy to use
- No prescription required




Questions | Order
800.438.8592
www.timmmedical.com

Erectile Dysfunction and Venous Leak

Venous leak is a condition where the veins in the penis do not keep blood in the penis during an erection. Men with this condition may not have a problem getting an erection, but because the blood does not stay in the penis, maintaining the erection can be difficult.

How it Works

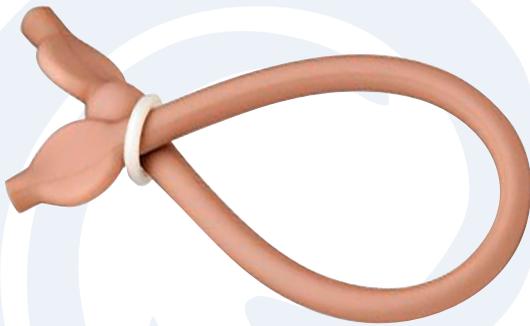

Venoseal™ is an Adjustable Constriction Loop that is placed and tightened at the base of the penis. With proper tension, the Venoseal™ device will be comfortable to the user but also provide enough additional pressure to prevent blood from flowing back into the body, thus maintaining an erection.

Please read these instructions before using the Venoseal™ Adjustable Constriction Loop. This device may cause injury if not used correctly.

INSTRUCTIONS FOR USE:

For optimum results, the Venoseal™ Adjustable Constriction Loop should be placed onto the penis before a full erection is achieved. Follow the instructions below to ensure proper use.

- 1. Stretch the Venoseal™ device:** Grasp the loop of the Venoseal™ with the left hand making sure that the ball is on the top and stretch the Venoseal™ loop.
- 2. Place the Venoseal™ around the shaft of the penis:** Place Venoseal™ so that the loop goes around the base of the penis. The part of the tubing with the ball should be on the top side of the penis.
- 3. Adjust tension:** Grasp the tab end (lower end) of the tubing without the ball. Pull horizontally away from the penis until Venoseal™ is gently snug. Venoseal™ should be just tight enough to keep the blood from leaving the penis. Too much tension may stop the erection process, may lead to discomfort, and could cause harm to the penis.

To Remove Venoseal™: Venoseal™ may be loosened by pulling up on the end of the tubing with the ball. Once loosened, Venoseal™ can be removed by sliding the loop over the end of the penis.