

How to Get Renew

RX ONLY



Step One

Discuss Renew with your provider and confirm it's appropriate.



Step Two

Your doctor* writes a prescription (Starter Package + 30-day supply).



Step Three

Prescription is verified, then shipped directly to you.



Step Four

After sizing, continue with a 30-day supply (30 disposable inserts with applicators).

**Valid Prescriptions can only be accepted by MD, DO, NP, or PA.*

Important Safety Information

- ✓ Do not use if allergic to silicone or using suppositories
- ✓ Stop use and contact your clinician if you experience bleeding, pain, or irritation
- ✓ Do not use if you've had rectal surgery within the past 4 weeks

Talk with your clinician

Use this guide to start the conversation.

Share

- How often leakage happens and what triggers it
- Urgency, stool consistency, and impact on daily life
- What you have already tried (diet, medications, pelvic floor therapy, routines)

Questions to ask

- Is Renew appropriate for my symptoms?
- Any conditions, surgeries, or symptoms that could make Renew inserts unsafe or less effective for me?
- What follow-up do you recommend after my sizing trial?

Contact Information



Location

14 Kendrick Road
Wareham, MA 02571



Phone

800.382.5879



Email

info@cmtmedical.com



Your Fecal Incontinence Solution Starts Here!

Renew Inserts


Confidence. Control. In Comfort.

Discreet internal protection for Accidental Bowel Leakage (ABL) / fecal incontinence.



How Renew Works

A soft silicone insert is placed in the rectum to create a temporary seal that reduces/eliminates involuntary stool leakage. It stays in place until the next bowel movement, then is removed and replaced.

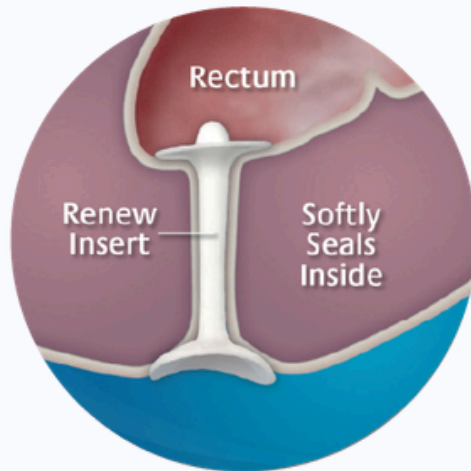

Wear Time: A single insert may be worn day and night until the next bowel movement, or changed as desired.

Key Features: Made from soft, flexible silicone for comfort, this device is a single-use insert with a hygienic fingertip applicator for easy placement.



Scan QR code

Find Renew easily on our website!

What is ABL or Fecal Incontinence?

- Accidental bowel leakage is the involuntary loss of bowel control, which may include passing solid or liquid stool or mucus. It can affect everyday confidence, work, travel, and social life. Many people delay care because it feels embarrassing. Your clinician can help.

Who may benefit?

- Renew is designed for patients with ABL/FI and may be considered when conservative management is appropriate.

You're Not Alone

(US Data)

Fecal Incontinence by the Numbers

- 1 in 12 adults (8.3%) experience monthly symptoms
- 1 in 37 adults (2.7%) experience weekly symptoms
- Women: ~1 in 11 (8.9%) | Men: ~1 in 13 (7.7%)
- Nearly 1 in 6 adults 70+ (15.3%)

Source: NHANES 2005-06 (Whitehead et al., Gastroenterology 2009).

Sizing & Starter Pack

Renew is available in two sizes: Regular and Large. First order a Starter Package to determine your best fit. Starter packages include one Regular and one Large insert, each with an applicator.

Sizing Tip:

Try Regular first. If it comes out unexpectedly or leakage persists, try Large.

Insertion Positions:

Try sitting on the toilet, standing with one foot up, or squatting—use the position that feels most comfortable.

